

# Oticon ♦ ConnectLine

## PRODUCT INFORMATION TV ADAPTER

*The Oticon ConnectLine TV adapter takes the TV sound and wirelessly transmits this in high quality to Streamer for optimal TV experiences. This dedicated solution for TV listening is easy to install and gives a high-end, natural sounding TV experience free of noise and artifacts. As part of the ConnectLine™ system, the TV application stays connected by the AutoConnect feature and works seamless and simultaneous with other applications such as home phone, mobile phone and music player.*

### **Premium TV listening**

ConnectLine TV listening sets a new standard for hearing instrument users. Both in audibility and satisfaction, since users will get more out of their hearing solution.

The TV sound arrives directly in the user's hearing instruments via Oticon Streamer with the original richness and nuances carefully accentuated. Because of the crystal clear sound and its natural response, the ConnectLine TV adapter will be most suitable for both mild and severe hearing losses.

### **Ease of use**

The TV adapter is easy to install and daily use with Streamer is both userfriendly and secure. The AutoConnect feature ensures that the ConnectLine TV adapter is always ready for use whenever within 10 m range from the user.

TV listening starts just by pressing the Audio button on Streamer. Indications on the TV adapter give clear status whether transmission is active or in stand-by.

### **Fashionable design**

Its slim, elegant design and inconspicuous appearance makes the TV adapter suitable to be placed either on the TV furniture or behind the TV set.


### **Simple Pairing**

The ConnectLine TV adapter automatically detects when a Streamer is in pairing mode and requests pairing. No need for pushing buttons or special power up routines, just place Streamer next to the TV adapter.



## Included Accessories

- Power supply
- TV cord 1,5 m  
(3.5 mm jack - 2 x RCA)
- TV interface adapters

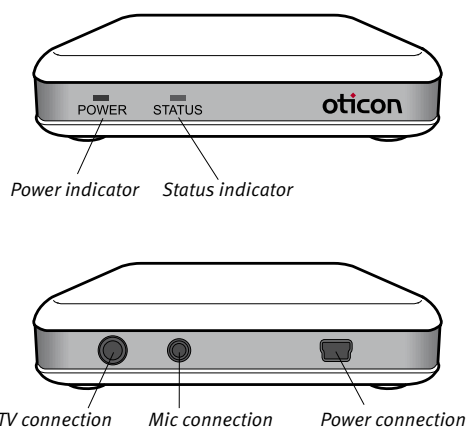

## Technical data TV 75-01

Dimensions	80 x 80 x 17 mm
Weight	70 g
Power supply	External wall adapter, 5 V dc
Power consumption	Max. 60 mA
Audio interfaces	Line input (3.5 mm jack) Mic input (2.5 mm jack)
Line input sensitivity	0,5 Vrms - 2,0 Vrms (analogue)
Compatibility	Streamer FW 1.2
AutoConnect	Yes (10 meters)
Simple Pairing	Yes
Transmission range	10 meters (Class 2)
Transmission latency	< 15 mSec
Audio Quality	Mono / 16 kHz sample frq
Audio Bandwidth	80 Hz - 7,4 kHz
Status indications	Green when powered Orange when transmitting
Colour	White, high gloss
Emissions	Max. -15dBµA / 10 meters In compliance with all relevant standards
Operating temperature	0 - 50 °C
Storage temperature	-20 - 70 °C

### Genie and ConnectLine TV adapter

Streamer compatible Oticon hearing instruments always have a program for the Streamer TV application.

The dedicated Streamer TV program is automatically activated by the hearing instrument as soon TV listening is started through Streamer.

The Streamer TV program cannot be fine tuned directly in Genie. The relevant amplification applied in this program is similar to the general fitting, using P1 as reference.

If fine tuning however is needed, an additional standard program is fitted to the instrument as P2, which then must be the user's standard listening program. P1 can now be fine tuned to the specific TV listening application.

During TV listening, the hearing instrument microphones are normally turned on (unless temporarily muted by the user). The sound level of the microphones (i.e. acoustic input from surroundings) can be attenuated relatively to the TV sound. This is done in the "End Session" of Genie. By default, the microphone level is set to -6 dB relative to the TV audio.